



To ensure the optimal use of the Fast Track lyophilised tubes in your thermocycler, please read carefully the following information:

DO ✓

DON'T ✗

Strips can be separated by cutting with scissors.



Whenever possible, you should avoid cutting in single tubes to ensure the optimal fitting accuracy in the precision holder.

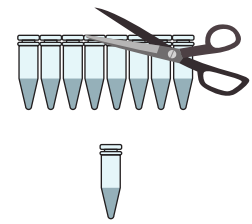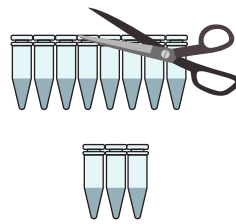



When using few tubes, always use "supporter" tubes on the left and on the right of the block to have constant pressure of the lid on all tubes.

○ reactions tubes

● supporter tubes (can be re-used between runs)

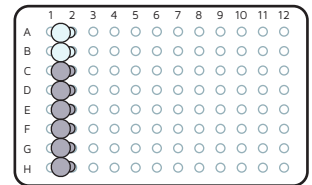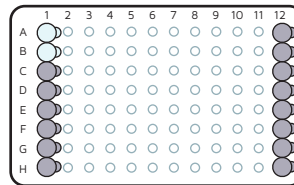

Always position strips vertically.

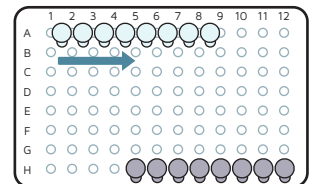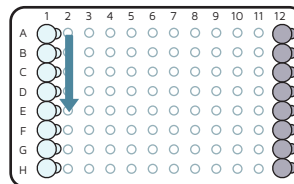

From the moment you have "supporter" tubes on each extremity, feel free to position your reactions tubes wherever you want to.

